

# Pediatrics\* Minimums for NutrEval



Blood Draw Tube	Volume of Blood Draw (mls)	Pediatric Minimum Draw Amounts (mls)	Processing?	Transfer Tube?	Biomarkers
Na EDTA	6	3	No	NA	Elemental
SST	8.5	3	Yes	Amber Transfer tube with Serum label	Vitamin D and CoQ10
EDTA	4	4	No	NA	Glutathione and EMFA

Urine Tubes are the same as current (SSA tube for amino acids; Thymol tube for OA; Neutral tube for 8OHdG)



Blood Draw Tube	Volume of Blood Draw (mls)	Pediatric Minimum Draw Amounts (mls)	Processing?	Transfer Tube?	Biomarkers
Na Heparin	6	6	Yes	Yellow Top Tube	Amino Acids (Plasma)
Na EDTA	6	3	No	NA	Elemental
SST	8.5	3	Yes	Amber Transfer tube with Serum label	Vitamin D and CoQ10
EDTA	4	4	No	NA	Glutathione and EMFA

Urine Tubes are the same as current (Thymol tube for OA; Neutral tube for 8OHdG)

\* NutrEval FMV and NutrEval Plasma testing is not available for those less than 2 years of age. Specimens for patients less than 2 years of age will be discarded



Call 800.522.4762 or visit our website at [www.gdx.net](http://www.gdx.net)